# Alcotest 7110 Standard IR / IR + EC

## Atemalkoholmessgerät

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Dräger Safety AG & Co. KGaA

## For Your Safety

### Strictly follow the Instructions for Use

Any use of the instrument requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

### Maintenance

The instrument has to be inspected by experts at least annually.  
Repair of the instrument may only be carried out by trained service personnel.  
We recommend that a service contract be obtained with DrägerService and that all repairs also be carried out by them.  
Only authentic Dräger spare parts may be used for maintenance.

### Not for use in areas of explosion hazard

The instrument is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

### Coupling with electrical instruments

Electrical coupling with instruments not mentioned in these instructions for use only with agreement of Dräger Safety AG & Co. KGaA.

### Liability for proper function or damage

The liability for the proper function of the instrument is irrevocably transferred to the owner or operator to the extent that the apparatus has been serviced or repaired by personnel not employed by DrägerService or if the instrument is used in a manner not conforming to its intended use.  
Dräger cannot be held responsible for damage caused by non-compliance with the recommendations given above.  
The warranty and liability provisions of the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

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## Intended use

Alcotest 7110 Standard is a measuring instrument for the determination of the breath alcohol concentration (BrAC).

It measures the ethanol concentration in the exhaled air.

It is intended to be used stationary as well as mobile in motor vehicles.

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## Functional description

### Measuring procedure

The Alcotest 7110 Standard performs a measuring cycle with a single breath sample to determine the breath alcohol concentration.

During the delivery of the breath sample by forced expiration the exhaled air is led through a mouth-piece into the instrument.

For the determination of the concentration an infrared optical sensor (Alcotest 7110 Standard IR) or a combination of an infrared optical and an electrochemical sensor with different analytical specificity (Alcotest 7110 Standard IR + EC) is used.

The necessary activities of the operator and of the person being tested are shown by short messages and instructions on the display of the Alcotest 7110 Standard. Only if all single steps of the breath alcohol determination are performed correctly and if none of the internal test criteria is outside of the admitted tolerance, the final result of the breath alcohol measurement is given and a protocol is printed on the internal printer.

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## Keyboard

### Keyboard operation

A keyboard may be connected to the Alcotest 7110 Standard.

The keyboard is used for:

- entering data of the person being tested for the protocol.
- entering and downloading of instrument parameters and defined wordings (as for example location, date).

For entering data into the instrument only a keyboard listed as accessory has to be used.

The use of other keyboards is not permissible without agreement of Dräger Safety AG & Co. KGaA.

The Alcotest 7110 Standard processes internally all letters as capital letters. Umlauts and "ß" may not be entered. Input corrections may be done by using the backspace button.

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### Keyboard inputs for instrument settings

Typical sequence using a keyboard for entering resp. changing instrument settings is:

- press key [ **ESC** ].

The following is displayed:

#### FUNCTION:

A code for the required function may now be entered.

The available functions are listed on page 16.

All inputs are finished by actuating key [ **ENTER** ] resp.

[ **RETURN** ].

If a selected function is not to be performed or is to be interrupted it may be stopped resp. interrupted by actuating key [ **ESC** ].

If within 2 minutes after calling a function no input occurs, the Alcotest 7110 Standard returns again to the normal ready status.

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## Transportation and Storage

Usual precautions should be taken for transportation of the instrument.

Excessive mechanical strains, shocks and vibrations should be avoided when transporting the instrument.

The instrument should be stored in dry rooms.

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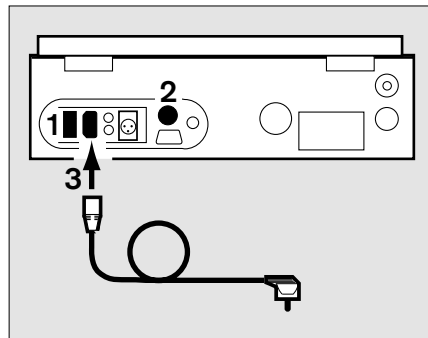
## Putting into operation

### Preparation of measurement

Choose the place the instrument such that the gas release on the rear of the instrument is not obstructed and that direct sun light on the instrument is avoided.

- Pay attention to sufficient space for accessories (keyboard, etc.).
- Place instrument on an even and solid surface.
- Remove the case cover.

### Stationary use

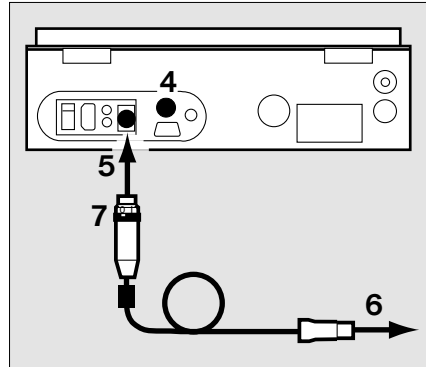
- Observe power supply voltage on the instrument plate.
- 1 Bring main switch into position "0" (OFF).
  - 2 Connect keyboard, if operation with keyboard (see accessories and page 4).
  - 3 Connect power cable.
- 1 Bring main switch into position "I" (ON).
- The Alcotest 7110 Standard begins with the warming-up phase.

### Mobile use

A minimum DC voltage of 11.5 V for battery operation (12 V connection) of the Alcotest 7110 Standard is required. At too low input voltage the instrument is switched off. An over-discharging of the supply battery is precluded.

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Therefore:

- ensure a good charging condition of the motor vehicle battery or provide an additional battery.
- 4 Connect keyboard, if operation with keyboard (see accessories and page 4).
- 5 Connect and secure 12 V connecting cable (see accessories).
- 6 Insert connector of the connecting cable into the 12 V socket of the motor vehicle or into the socket of the cigarette lighter (with adapter).
- 7 Bring switch of the 12 V cable into position "I".

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The Alcotest 7110 Standard begins with the warming-up phase.  
(The main switch of the instrument is not active at 12 V operation.)

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## Ambient conditions for measurement

Ensure compliance with the admissible ambient conditions (page 25).

The Alcotest 7110 Standard IR + EC tests the ambient air for possible influences on the result. If the instrument detects such influences (message in the display: » **ALKOHOL IN THE ENVIRONMENT** « or » **BLANK CHECK INCORRECT** «), a measurement is not possible. In case of such a message measures have to be taken to improve the ambient air (airing, different place of the instrument, etc.).

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**STARTBEREIT 03.02.01 08:45**

Akustisches Signal: 2 x Piepton

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## Establishing readiness for operation

### Warm-up phase and self-test

After switching on the instrument a self-test is performed which takes approximately 30 seconds. The faultless function of the instrument is shown with the following display:

**AUTOTEST OK**

Audible signal: 1 x beep

Then the following is displayed:

**WARMING UP**

**NOT READY**

Afterwards the location is displayed:

**LOCATION: CLINIK HAMBURG**

additionally date and time are shown:

**03.02.01 08:30 W**

S = summer time, W = winter time

During the warming-up phase all instrument components which come into contact with the breath sample are heated to above 40 °C to avoid any condensation from the breath air in the instrument.

Until readiness for measurement is reached the following is displayed alternately:

**WARMING UP**

**NOT READY**

The duration of the warm-up phase is normally less than 15 minutes. At low temperatures this phase is elongated, but may be shortened by covering the instrument with the case cover. If the warming-up phase is not finished after 30 minutes, a fault message will be given.

When readiness for measurement is reached the following is displayed:

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When for 15 minutes no measurement has been performed, the instrument switches into a stand-by mode for saving power displaying:

**STAND-BY (READY → PUSH BUTTON)**

From this mode the readiness for measurement may be obtained again in approx. 3 minutes.

Basically the Alcotest 7110 Standard is suitable for continuous operation, but it should be switched off in case of operation interruptions of more than one week.

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## Making measurement

### Conditions

The measurements of the breath alcohol concentration with the Alcotest 7110 Standard are to be performed only by persons trained with the instrument!

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### Control time before measurement

Basically for measuring breath alcohol the following has to be considered:

- Within a period of at least 10 minutes before the measurement the intake of substances through mouth or nose has surely to be precluded. Actual residues of alcohol may give false measurements. This may also occur by aromatic drinks (e.g. fruit juices), alcoholic mouth sprays, medicines and drops, as well as by burping and vomiting.

**Rinsing out the mouth with water or non-alcoholic drinks do not substitute for an interval!**

### Starting measurement

- Press start button

If the instrument is in the stand-by mode shown by » **STAND-BY (READY → PUSH BUTTON)**«, push the start button firstly to obtain instrument readiness for measurement.

Before the measurement the data of the person being tested may be entered depending on the instrument setting (page 16).

The input of respective data has to be finished and acknowledged again by pushing the key [ **ENTER** ].

The first input request is:

**NAME SUBJECT:**

The name of the person being tested has to be entered via the keyboard. Up to 23 characters may be entered for the name. For names with more than 23 characters the last character is accepted at the 23th position. The input has to be finished with the key [ **ENTER** ], the entered name is shown again and has to be acknowledged once more.

Afterwards the display shows:

**FIRST NAME SUBJECT:**

As second information the first name of the person being tested has to be entered with again up to 23 characters. The input is also finished with the key [ **ENTER** ] and has to be acknowledged again.

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As third information the date of birth of the person is requested with the display showing:

**DATE OF BIRTH SUBJECT: DD/MM/YY**

For this input only the digits for the day of birth (DD), the month of birth (MM), and the year of birth (YY) have to be entered.

- Two digits have to be entered respectively for the day, the month and the year (e.g. » 04 « for month April). Internally the data input is checked for consistency concerning day and month. The 31.04.56 would for example not be accepted. For input of a non reasonable date the acceptance of the date after pushing the key [ ENTER ] is refused and a new input of date is requested.

After entering the data of the person being tested the display shows:

**DATE ENTRIES READY**

So far all previous inputs may be called back again on the display by pushing the key [ ESC ]. It may be changed by a new input or may be acknowledged by pushing the key [ ENTER ].

**When pushing the key [ENTER] as acknowledgement of the display shown above the measurement cycle begins immediately – a return to the previous inputs by the key [ ESC ] is no longer possible.**

When finishing the data input the measurement procedure is continued with the display showing:

**PURGING**

In the Alcotest 7110 Standard IR + EC the air drawn in is analysed for alcoholic vapours. The display shows:

**ENVIRONMENT CHECK**

If influences of the ambient air are excluded due to this test the zero value is determined shown by:

**BLANK CHECK**

The Alcotest 7110 Standard now requests for a breath sample:

**PLEASE BLOW >**

Audible signal: 1 x beep

Open the packaging of a new mouth-piece (see Order List), remove the sampling hose from the storage recess, and insert the mouth-piece into the handle at the end of the sampling hose without skin contact by using the packaging.

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## Delivering breath samples

The operator of the instrument has to ensure that during the control time (at least 10 minutes) before the delivery of the breath sample the person being tested did not take in any substances through mouth or nose, and that the behaviour of the person is not conflicting with the successful delivery of a breath sample.

When the display starts to show » **PLEASE BLOW** « the instrument is ready for measurement for 2 minutes. Within this time period the person being tested has to deliver the breath sample.

- Request the person being tested to inhale calmly and then to blow continuously and evenly into the mouth-piece until all asterisks are shown in the display.

During the delivery of the breath sample the asterisks are shown in the display according to the progress. The length of the still empty asterisk line is an indication for the volume still to be delivered until the minimal volume is reached.

**PLEASE BLOW > \* \* \* \* \***

Even if all asterisks are already shown in the display, the delivery of the breath sample may be continued. The measurement will not be influenced by this.

When breath flow falls below the minimal flow at the end of the delivery of the breath sample the display shows:

**STOP**

The delivery of the breath sample has to be stopped immediately.

**A reblowing has absolutely to be avoided!**

If the requirements for the delivery of the breath sample are fulfilled concerning duration, volume and continuity, the mouth-piece has to be removed from the handle at the end of the sampling hose after the request

**REMOVE MOUTHPIECE**

using the packaging of the mouth-piece and has to be disposed. The breath hose has to be placed into the storage recess or besides the instrument.

The measuring system is purged again and a further internal alignment is performed shown by:

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Afterwards the final result is shown in the display if all requirements of the procedure are fulfilled.

Example:

<b>RESULT</b>	<b>0.46 mg/l</b>
---------------	------------------

Measurement results below 0.025 mg/l resp. 0.05 ‰ are given as 0.00 mg/l resp. ‰.

The print-out of the protocol starts.

### Finishing the measuring cycle

After the end of the print-out the display shows:

<b>COPY? PRESS BUTTON PLEASE</b>
----------------------------------

This possibility to print a further protocol is shown in the display for 10 seconds. By pressing the key [ **ESC** ] the procedure may continued immediately.

If no copy has been required the instrument finishes the measuring cycle finally.

The instrument returns to readiness for operation for the next measuring cycle shown by:

<b>READY</b>	<b>03.02.01 08:55</b>
--------------	-----------------------

Audible signal: 2 x beep

### Printing measurement protocol

At a later time before the next measurement has been started a copy of the last print-out may be initiated by pushing twice (double click") the paper-feed button (page 18).

After the double-click the display shows for 10 seconds:

<b>COPY? PRESS BUTTON PLEASE</b>
----------------------------------

Pushing the start button starts the print-out. After switching off the instrument the possibility does no longer exist.

An afterwards initiated print-out contains all data of the measurement.

### Failings during measurement cycle

At each time of the measurement cycle a considerable number of instrument controls is performed, each breath sample is checked concerning certain criteria to see whether it is usable and correctly delivered.

If failings in the measurement cycle are detected, a corresponding remark appears on the display and depending on the failing the breath sample has to be delivered again or the measurement cycle is stopped completely.

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If for example the exhaled breath volume is too small the display shows:

#### MINIMUM VOLUME NOT ACHIEVED

The delivery of the breath sample has to be repeated. (Totally 2 unsuccessful attempts are admitted for one measuring cycle.)

The repetition of the breath sample is noticed by:

#### TEST REPEATED

The instrument becomes automatically ready for measurement again.

Identically the measuring cycle is continued, if e.g. due to a too short delivery of the breath sample (less than 3 seconds) the remark

#### BLOWING TIME TOO SHORT

is shown.

For breath samples with a too small exhalation volume as well as a too short exhalation time the fault message concerning the breath volume is shown.

After a third insufficient attempt the measuring cycle is stopped with the remark:

#### TEST ABORTED

and a corresponding remark is printed on the protocol.

If the presence of mouth alcohol is detected in the breath sample by monitoring the concentration over time the display shows:

#### MOUTH ALCOHOL

The measurement cycle is stopped immediately and a protocol with a respective remark is printed.

If within 2 minutes after the request » **PLEASE BLOW** « the delivery of the breath sample is not started, the measurement cycle is also stopped and the display shows:

#### READINESS TO BLOW EXPIRED

A protocol with a respective remark is printed.

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## Explanations of the protocol

The information given in the upper part of the protocol enables the unambiguous identification of the measuring instrument used for the measurement and which printed the protocol.

- Serial number: unambiguous information (4 characters, 4-digit number) exactly related to one instrument.
- Test number: number of measurements with print-out of a protocol after the last service at which the numerator has been zeroed.
- Calibration number: control number which is changed when adjusting the instrument.

The other information in the protocol is self-explaining.

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## Instrument settings, instrument downloads

### Overview

The Alcotest 7110 Standard offers to the operator several functions to show data or to select settings.

It is not possible to call up the functions during the sequence of a measuring cycle.

The selection and the use of these functions via the keyboard is explained on pages 4 to 5.

The following list contains the functions available for the operator of the Alcotest 7110 Standard.

Input (code word)	Function/Reaction of the instrument
» LOCATION «	Input of location or operator of the instrument. The information is printed on protocol.
» TIME «	Date and time with seconds are shown on the display for 10 seconds.
» DATE «	Date and time with seconds are shown on the display for 10 seconds.
» OFF «	The instrument is switched off "electronically". By pushing the start button it may be switched on again.
» CHANGE - TIME «	The switching between summer time and winter time is performed.
» COPY «	Possibility to print out a further protocol of the last measurement.
» VERIFY «	Shows the expiry date of the verification interval (month and year).
» MINUTE «	Change of the actual time by +/- 1 minute (possible only once per day).
» STAND - BY «	The instrument is switched into the stand-by mode. After pushing the start button it is becoming ready again.
» LANGUAGE «	Selection of instrument language possible.
» SET - CLOCK «	Input of time and date.

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Input (code word)	Function/Reaction of the instrument
»PRINTOUT«	<ol style="list-style-type: none"> <li>1. Input of number of printed protocols.</li> <li>2. Selection of data input of the person being tested (selection by blank key).</li> <li>3. Input of test number.</li> </ol>

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im Display angezeigt.

### Switching summer time / winter time

The respective time chosen in the Alcotest 7110 Standard is noticed by the additional character at the time information. For switching keep start button pressed during switching on the instrument until the following is displayed:

**SUMMER TIME - WINTER TIME**

The time is set automatically one hour forward resp. backward, the respective additional character is chosen.

Note: Switch over cannot be performed between 23.00 hours and 01.00 hours.

### Interruption of measurement - Quick Reset

The measuring cycle can be stopped by keeping the start button pressed at least 2 seconds.

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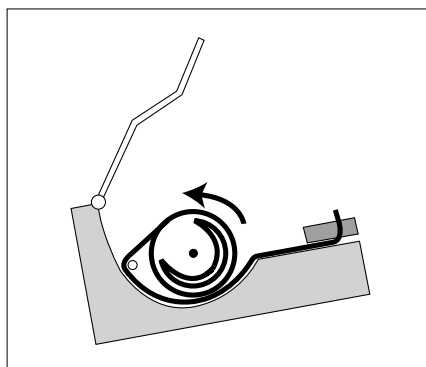
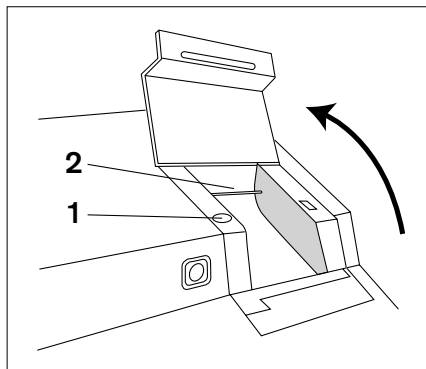
## Service

### Check

Inspections of the Alcotest 7110 Standard have to be performed by experts, e.g. by DrägerService or persons authorised by DrägerService

#### Maintenance intervals:

recommended: 6 months  
maximal: 12 months

### Inserting paper

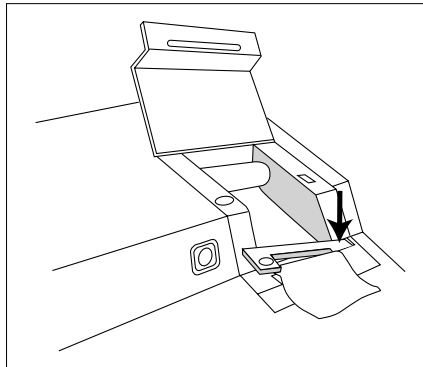
After start of the measurement the Alcotest 7110 Standard checks whether sufficient paper is in the printer. Not sufficient paper is displayed with the message » **NO PRINTER PAPER** «.

**When a red strip appears on the paper a new paper roll has to be inserted.**

- Open flap of printer.
  - 1 Remove remaining paper by pressing red paper feed button.
  - 2 Feed start of paper behind the metal guide and push it into the slot of the printing mechanism.
  - 1 Keep paper feed button pressed until paper is transported several centimetres through the printing mechanism.
  - Paper has to run between typewriter ribbon and ribbon cassette.
- 
- Position paper roll in the recess.
  - Insert start of paper from below through the slot in the flap, close flap.

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
### Changing ribbon cassette

- Open flap of printer.
- Remove consumed cassette: press ribbon cassette on spot marked with "PUSH", take cassette out.
- Tension ribbon of new cassette by turning toothed knob clockwise.
- Insert and press on new cassette.
- Paper has to run between type-writer ribbon and ribbon cassette.
- Close flap.

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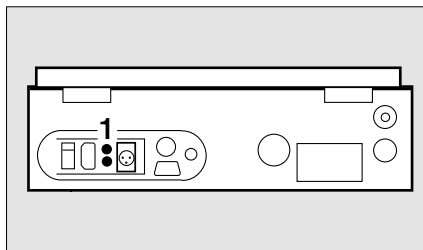### Replacing fuses

- Use screwdriver with appropriate blade.
- 1 Remove both fuse tops.
- Replace old fuses by new ones. Only use the prescribed fuses, page 26.
- 1 Close both fuse tops again.

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### Cleaning instrument

- Disconnect instrument from the power supply.
- Wipe over surfaces of instrument and hose with a damp cloth.
- Dry thoroughly with a dry cloth.

Do not use any cleaning agents containing alcohol!  
Do not allow any liquid to ingress into the hose.

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### Disposal of the instrument

- at the end of its service life.
  - Dispose of the instrument in accordance with national waste disposal regulations or
  - Ask a suitable disposal contractor to dispose of the instrument.
- The local environmental agency can supply further details.

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## Fault, Cause and Remedy

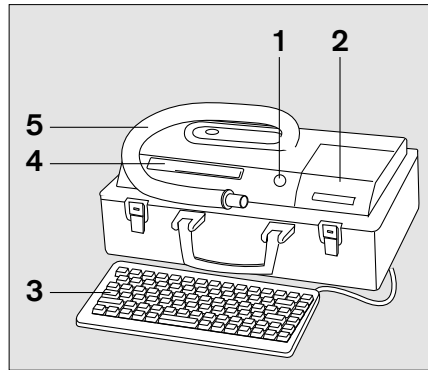
Fault / Display (Printer text)	Cause	Remedy
No function after switching instrument on	Fuses eventually defective.	Check fuses, replace.
	Power supply insufficient.	Check power supply.
BLANK CHECK INCORRECT (BLANK CHECK INCORRECT)	Zero adjustment incorrect.	Ensure that ambient air is clean and free of alcohol. Air test room.
MOUTH ALCOHOL (MOUTH ALCOHOL)	Residual alcohol in the mouth of the person being tested. Person have had alcohol immediately prior to measurement or had belched.	Remedy cause. Repeat measurement after waiting time. Observe control time.
MINIMUM VOLUME NOT ACHIEVED (MIN. VOL. NOT ACHIEVED)	The supplied volume is smaller than the minimum volume required.	The instrument automatically requests to repeat the breath sample.
BLOWING TIME TOO SHORT (BLOWING TIME TOO SHORT)	The time of the supplied breath sample is shorter than the minimum time required.	The instrument automatically requests to repeat the breath sample.
BLOWING NOT ALLOWED (NOT ALLOWED BLOWING)	Delivery of the breath sample at a not permitted time; no readiness for operation. (Reblowing)	Remind the person to be tested of the correct delivery of the breath sample. Repeat measurement.
READINESS TO BLOW EXPIRED (READIN. TO BLOW EXP.)	Readiness to blow of 2 minutes is expired. No delivery of breath sample within this time.	Repeat measurement.
ALCOHOL IN THE ENVIRONMENT (ALCOHOL-ENVIRONMENT)	Contamination in the ambient air is detected.	Ensure that ambient air is clean and free of alcohol. Air test room.
FUNCTION NOT POSSIBLE	Incorrect input of a code word.	Check code word. Repeat input.
PURGING ERROR (PURGING ERROR)	Necessary volume for purging the measuring system not reached.	Check inlet of the breath hose. If no conspicuousness, repeat measurement. If fault message reappears: inform DrägerService.
INTERFERENCE (INTERFERENCE)	The two independent measuring systems (only Alcotest 7110 Standard IR + EC) have detected an inadmissible influence on the measured values by foreign substances in the breath.	Identify cause of the influence, remedy, if possible. Eventually initiate other evidential measures.

Fault / Display (Printer text)	Cause	Remedy
<p>ERROR [XXX]</p> <p>..MAIN-SYSTEM ..IR-SYSTEM ..EC-SYSTEM ..FLOW-SYSTEM ..PRESSURE-SYS ..HEATER-SYSTEM (Message is printed out additionally)</p> <p><b>Example:</b> ERROR [075] HEATER-SYSTEM</p>	<p>Instrument defective (XXX = number specifying the error)</p> <p>Instrument blocks automatically all measurement sequences when error is detected.</p> <p>The predetermined temperature of the sensor or the hose is not reached.</p>	<p>Switch off instrument. Check whether the error message reappears after switching on again. If yes: inform DrägerService. Bring instrument to repair.</p> <p>Ensure compliance with the ambient conditions. Switch instrument off, eventually close the instrument cover and switch on again.</p> <p>If the error message reappears after switching on, inform DrägerService.</p>

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
## What's What
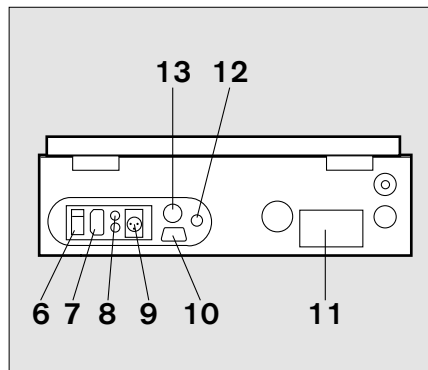
### Front view

- 1 Start button
- 2 Printer flap / Paper roll /  
Paper feed button
- 3 Keyboard
- 4 Display
- 5 Heated sampling hose in the  
storage recess

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01123832

### Rear view

- 6 Mains switch
- 7 Connector for power supply
- 8 Fuses
- 9 Connector 12 V
- 10 RS 232 connector
- 11 Type plate
- 12 Socket hard key plug
- 13 Connector keyboard

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### ⚠ Warning:

#### Connector 12 V

Only batteries or DC power supplies with mains separation (for example according to VDE 0551) have to be connected.

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### Ambient conditions

#### For operation

Temperature	0 °C to 40 °C
Ambient pressure	850 to 1050 hPa

#### For storage

Temperature	-25 °C to 60 °C
-------------	-----------------

### Measuring system

#### Measurement range

Breath alcohol concentration	0 to 3 mg/l 0 to 300 µg/100 ml (Ethanol mass per breath volume at 34 °C and 1013 hPa)
Blood alcohol concentration	0 to 6 ‰ 0 to 6 g/l 0 to 0.6 % (Ethanol mass per blood volume at 20 °C and 1013 hPa. By giving this data a conversion factor of 2100 between breath and blood alcohol concentrations is used.)

Resolution final result	0.01 mg/l 1 µg/100 ml 0.01 ‰ 0.01 g/l 0.001 %
-------------------------	---

Duration of measuring cycle (typically)	2 minutes
---	-----------

Blowing time	>3 seconds
--------------	------------

Volume	> 1.5 l
--------	---------

#### Repeatability (according to DIN VDE 0405-2)

	standard deviation smaller than
0 to 0.4 mg/l	0.006 mg/l
0 to 40 µg/100 ml	0.6 µg/100 ml
0 to 0.8 ‰	0.012 ‰
0 to 0.8 g/l	0.012 g/l
0 to 0.08 %	0.0012 %

above 0.4 to 1.0 mg/l	variation coefficient
above 40 to 100 µg/100 ml	smaller than 1.5 %
above 0.8 to 2.0 ‰	
above 0.8 to 2.0 g/l	
above 0.08 to 0.20 %	

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**Order List**

Name	Part no.
Alcotest 7110 Standard IR	83 15 080
Alcotest 7110 Standard IR + EC	83 15 800
<b>Consumables and Accessories</b>	
Keyboard "standard"	83 13 215
Keyboard "compact"	83 15 497
12 V cable (with ON/OFF switch)	83 12 641
Printer paper ( 5 rolls)	81 00 318
Ribbon cassette	81 00 284
Mouth piece with non-return valve (25 pieces)	68 05 703

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**Index**

<b>Accessories</b> .....	28
Ambient conditions .....	25
Ambient conditions for measurement .....	7
Areas of explosion hazard .....	3
<b>Battery operation</b> .....	6
<b>CE marking</b> .....	26
Changing ribbon cassette .....	19
Check .....	18
Cleaning instrument .....	19
Code word .....	5
Conditions .....	10
Consumables .....	28
Control time .....	12
Control time before measurement .....	10
Coupling with electrical instruments .....	3
<b>Data of the person</b> .....	10
Delivering breath samples .....	12
Determination of the concentration .....	4
Dimensions .....	26
Disposal of the instrument .....	19
Drift of the measurement result .....	26
Duration of measuring cycle .....	25
<b>Entering data</b> .....	4
Establishing readiness for operation .....	8
Explanations of the protocol .....	15
<b>Failings during measurement cycle</b> .....	13
Fault, Cause and Remedy .....	22
Finishing the measuring cycle .....	13
Front view .....	24
Functional description .....	4
Fuses .....	26

<b>Haftung</b> .....	3	<b>Input (code word)</b> .....	16
<b>Inbetriebnahme</b> .....	6	<b>Inserting paper</b> .....	18
<b>Instandhaltung</b> .....	3	<b>Instrument downloads</b> .....	16
<b>Instandhaltungsintervalle</b> .....	18	<b>Instrument settings</b> .....	16
<b>Kontrollzeit</b> .....	12	<b>Intended use</b> .....	4
<b>Kontrollzeit vor der Messung</b> .....	10	<b>Interruption of measurement</b> .....	17
<b>Konzentrationsbestimmung</b> .....	4	<b>Keyboard</b> .....	4
<b>Kopplung mit elektrischen Geräten</b> .....	3	<b>Keyboard inputs</b> .....	5
<b>Lagertemperatur</b> .....	25	<b>Keyboard operation</b> .....	4
<b>Lagerung</b> .....	5	<b>Liability</b> .....	3
<b>Leistungsaufnahme</b> .....	26	<b>Maintenance</b> .....	3
<b>Messablauf</b> .....	4	<b>Maintenance intervals</b> .....	18
<b>Messbereich</b> .....	25	<b>Making measurement</b> .....	10
<b>Messbereitschaft</b> .....	8	<b>Measurement protocol</b> .....	15
<b>Messergebniss</b> .....	13	<b>Measurement range</b> .....	25
<b>Messprotokoll</b> .....	15	<b>Measurement result</b> .....	13
<b>Messprotokoll ausdrucken</b> .....	13	<b>Measuring procedure</b> .....	4
<b>Messsystem</b> .....	25	<b>Measuring system</b> .....	25
<b>Messung abbrechen</b> .....	17	<b>Mobile use</b> .....	6
<b>Messung auslösen</b> .....	10	<b>Mouth-piece</b> .....	11
<b>Messung durchführen</b> .....	10	<b>Operating temperature</b> .....	25
<b>Messung vorbereiten</b> .....	6	<b>Order List</b> .....	28
<b>Messvorgang</b> .....	11	<b>Overview</b> .....	16
<b>Messzyklus beenden</b> .....	13	<b>Power consumption</b> .....	26
<b>Mobiler Einsatz</b> .....	6	<b>Power supply voltage</b> .....	26
<b>Mundspülung</b> .....	10	<b>Preparation of measurement</b> .....	6
<b>Mundstück</b> .....	11	<b>Print-out</b> .....	13, 15
<b>Nachatmung</b> .....	12	<b>Printing measurement protocol</b> .....	13
<b>Papierrolle einsetzen</b> .....	18	<b>Putting into operation</b> .....	6
<b>Probandendaten</b> .....	10	<b>Quick Reset</b> .....	17
<b>Quick Reset</b> .....	17		

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© Dräger Safety AG & Co. KGaA
der Schweiz****Dräger Austria GmbH**
**Dräger (Schweiz) AG**